

WART TREATMENT PEN

CAREFULLY READ THE INSTRUCTIONS FOR USE PRIOR TO USING ACETOCAUSTIN WART PEN, EVEN IF YOU USED THE PRODUCT BEFORE. KEEP THESE INSTRUCTIONS FOR USE IN CASE YOU NEED TO CONSULT THEM AGAIN. FOR ADULTS AND CHILDREN (4 YEARS AND OVER)

WHAT IS DR. YGLO WART TREATMENT PEN USED FOR?

Dr. Yglo Wart Treatment Pen is a medical device for the treatment of common warts on hands and feet. Acetocastin Wart Pen contains monochloroacetic acid, a substance which has a keratolytic effect on the skin. The wart will dissolve, layer by layer. Monochloroacetic acid ensures that the cells containing the wart virus are destroyed.

The benefit of using Acetocastin Wart Pen is that an application directly onto the wart does not hurt, which means it can also be used for children, when applied correctly by an adult.

Dr. Yglo Wart Treatment Pen can only be used on common warts, characteristics are:

- Primarily occur on hands and feet
- Same colour as the skin
- Circumference varies from less than a millimetre to more than a centimetre
- Several warts usually occur simultaneously
- Contamination often occurs through direct contact, but also through towels or infected surfaces
- Scratching and nail biting increase the risk of contamination



Do not use Dr. Yglo Wart Treatment Pen one senile warts

- Can come in different sizes and colours
- Average circumference is 0.5 to 1 cm
- Often start as a skin-coloured, yellow brown, greyish-brown patch. Later on, become bigger and fatter
- Appear as if they are lying on the surface of the skin and are easy to scratch off, but are actually firmly attached to the skin
- They can occur anywhere on the body, except for the palms of the hands and soles of the feet, and the lips. Most likely to occur on the chest, the back, the face, and the neck



Plantar wart (verruca plantaris)

- Found on the bottom/sole of your foot;
- Usually flat and surrounded by rough, thickened skin;
- Resemble calluses;
- Can contain black dots or tiny seeds;



Instructions for use (Prevent burns):

Do's:

- Application by adults;
- Use only on hands and feet! Do not use on other body parts;
- Rub Vaseline or zinc ointment into the surrounding skin to protect the skin;
- Dab the wart just once when applying;
- Treat the wart just once per week as per how to use;
- Allow the liquid to come into the contact with the wart/verruca only;
- Avoid contact with the healthy skin;
- Allow the liquid to dry properly after applying;
- Leave the wart/verruca uncovered;
- The result is visible after 1 week;
- Application to children shall be performed by an adult;
- Keep out of the reach of children.



DON'TS:

- Apply to warts/verruca on any part of the body other than the hands or feet;
- Apply to any wart that is wholly or partially under a nail;
- Apply on warts that occur completely or partially on the sole of the feet (plantar warts);
- Apply when your hands are shaking or your fingers are stiff;
- Dab on wart more than once when applying;
- Cover the wart/verruca after applying the liquid;
- Apply on birthmarks, water warts, open wounds, hyperpigmentation, senile warts, plantar warts, dark-coloured hairy or weird looking warts;
- Apply during pregnancy or lactation.
- Repeat treatment without a one-week interval.



CAUTIONS

Do not use Dr. Yglo Wart Treatment Pen:

- Under the age of 4 years old
- If you are sensitive to one or more of the ingredients.
- In the face (the fluid should never make contact with the eyes or the mucous membranes of the nose or mouth)
- Avoid contact with eyes. Contact with eyes may cause blindness
- Do not rub or touch eyes with hands if the product has been applied to them. If product gets into eyes, call medical help right away.
- For genital warts or in the genital area
- On birthmarks, molluscum contagiosum, open wounds, moles, naevus, senile (brown) warts, plantar warts, darkly coloured, hairy or strange looking warts
- On red, irritated, or infected warts
- Immediately after a surgical removal of warts, freezing of wart (cryotherapy) or a laser treatment. This particularly applies when the skin is red or painful after the treatment. In this case you will have to wait for at least 4 weeks, until the redness or inflammation of the skin has fully disappeared. Only then you can start using Acetocastin Acid Wart Treatment again
- During pregnancy or while breastfeeding
- On warts which occur completely or partly underneath the nail
- Do not use on warts that occur completely or partially on the sole of the feet (plantar warts).
- Having poor blood circulation: The treated area in diabetic patient and those with poor blood circulation may not heal as quick or completely.

Be extra careful when using Dr. Yglo Wart Treatment Pen:

- In case of doubt concerning the diagnosis (the change in the skin cannot be clearly explained as a wart)
- In case of immune deficiency (e.g. if you are using cytostatic medication)

In these cases, you may not use Dr. Yglo Wart Treatment Pen before a doctor has been consulted

WARNINGS

- Only intended for external use
- Use with utmost care. The fluid of Dr. Yglo Wart Treatment Pen is corrosive
- If the healthy skin makes contact with the fluid, please rinse it with water. If, despite these precautions, the skin becomes irritated, it will have to undergo the same treatment as a burn
- Contact your doctor if your complaints do not subside
- Properly close the pen after use and store it in an upright position
- Keep out of reach of children
- Use by children, the elderly and less able is not recommended
- Avoid contact with clothes, textile, table-cloths or skin
- Stop using if blisters occur
- Treat wart no more than once and repeat only if necessary after a one-week interval.
- The product should not be used beyond the expiration date.
- The lot number and the expiration date appear on the pen as well as the outside of the outer packaging.
- Do not use on multiple people as cross contamination of the virus can occur.
- Wash hands (except treated area) immediately after use.
- Inspect carefully before each use. If the product appears damaged, stop using and throw it away.
- Damaged product may leak fluid.

SIDE EFFECTS

Dr. Yglo Wart Treatment Pen may cause some side effects, although not everyone will experience these side effects. The side effects that could occur when using Dr. Yglo Wart Treatment Pen are irritated skin, burning sensation, skin redness or oversensitivity reactions. If any of these side

effects becomes serious, or if you notice any side effects not listed in these instructions for use, please consult your doctor or pharmacist.

INTERFERENCE WITH MEDICATIONS

As far as we know, there are no interactions with any other medications.

CONTENTS OF DR. YGLO WART TREATMENT PEN

A pen filled with 3ml a mixture of Monochloroacetic acid (50%) and purified water (50%).

STORAGE CONDITIONS

Store Dr. Yglo Wart Treatment Pen at a room temperature 15-25°C in the original container and out of reach of children.

The expiry date is stated on the bottom of the packaging.

Any serious incident that has occurred in relation to this product should be reported to the manufacturer and to the competent authority of the Member State in which the user is established.

For more information about Dr Yglo Wart Treatment Pen visit www.dryglo.com. For all product enquiries, please contact the distributor.

EXPLANATION OF SYMBOLS

The warning signs and symbols on the packaging and instructions for use are essential to ensure that you use this product safely and correctly and to protect you and others from injury. The symbols are explained in the table below:

Symbol	Explanation
	Symbol for the manufacturer
	Symbol that indications to consult the instructions for use
	Symbol that indicates that caution is necessary when operating the device
	Symbol for the expiration date
	Symbol that indicates that the item is a medical device
	Warning: Corrosive substance
	Not for children aged 3 years and younger
	Keep out of reach of children
	Not for pregnant women or while breastfeeding
	Symbol for the Lot number
	The product shall be stored between 15 °C and 25 °C
	Symbol that indicates the Unique device Identifier

HOW TO USE DR. YGLO WART TREATMENT PEN (APPLICATION ONLY BY ADULTS):

Please apply Dr. Yglo Wart Treatment Pen with utmost care. The solution is caustic. Be extra careful when applying it to parts on your hands or feet that are more difficult to reach. This particularly applies to persons with a limited mobility or in case of disturbed fine motor functions in the system (trembling hands, stiff fingers).



1. Before starting, protect the healthy skin around the wart by applying a thick layer of Vaseline or zinc ointment.
2. Hold the Acetocastin Pen in a slanted position, as if writing. Do not press or squeeze the pen.
3. Place the tip gently on the wart.
4. Apply the solution twice, each time for about 3 seconds (count slowly: 21, 22, 23).
5. Make sure the solution is applied only to the wart and not to surrounding skin.
6. After application, allow the solution to dry for about 5 minutes.
7. Once dry, carefully remove the Vaseline or ointment around the wart.
8. Close the pen tightly after use and store it securely, out of reach of children.
9. Do not cover the wart after application.
10. Do not treat the same wart more than once per week.

The fluid will need a week to dissolve the top layer of the wart, so you will not see an immediate result after application. Depending on the size and thickness of the wart, it may be necessary to repeat the treatment with Dr. Yglo Wart Treatment Pen after one week. The average treatment duration is around 5 weeks. Do not repeat treatment within a one-week interval.



APPLY ON COMMON WARTS ON HANDS AND FEET ONLY

4+ AGE USE ONLY



WARNING: PREVENT BURNS: CONSULT INSTRUCTIONS FOR USE BEFORE USING!

MANUFACTURER
TheOTClab B.V.
Fred. Roeskestraat 115,
1076 EE Amsterdam,
The Netherlands

Distributor
TheOTClab B.V.
Fred. Roeskestraat 115,
1076 EE Amsterdam,
The Netherlands