

WART REMOVAL TREATMENT

CAREFULLY READ THE INSTRUCTIONS FOR USE PRIOR TO USING ACETOCAUSTIN WART REMOVAL PLASTER, EVEN IF YOU USED THE PRODUCT BEFORE. THE INFORMATION MAY HAVE BEEN AMENDED ACCORDING TO NEW RESEARCH AND FINDINGS IN THE TIME SINCE YOU OPENED YOUR LAST PACKAGE. KEEP THESE INSTRUCTIONS FOR USE IN CASE YOU NEED TO CONSULT THEM AGAIN. FOR ADULTS AND CHILDREN (4 YEARS AND OVER)

WHAT IS ACETOCAUSTIN WART REMOVAL PLASTER USED FOR?

Acetocaustin Wart Removal Plaster is a medical device for the treatment of warts and verrucae. Acetocaustin Wart Removal Plaster contains 40% salicylic acid, a substance which has a keratolytic effect on the skin. The wart will dissolve, layer by layer. Salicylic acid ensures that the cells containing the wart virus are destroyed.

The benefit of using Acetocaustin Wart Removal Plaster is that an application directly onto the wart does not hurt, which means it can also be used for children, when applied correctly by an adult.

1. HOW TO USE: (APPLICATION ONLY BY ADULTS)

- Soak the wart in warm water for 5 minutes before application.
- Clean and dry the skin before use, and ensure the area is free of creams and oils.
- Take out one plaster, remove the foil from the back, avoiding touching the adhesive side.
- Apply the plaster with the salicylic acid center directly on the wart, ensuring the edges are smooth.
- Leave it in place until it starts to loosen. After a maximum of 48 hours, you must remove the patch.
- Soak the affected area in warm water for 5 minutes before you remove loose skin.
- Repeat this procedure as needed for up to 14 days.

⚠ CAUTIONS

Do not use Acetocaustin Acid Wart Treatment:

- Under the age of 4 years old by medical supervision
- Do not use if you are a diabetic person, hypersensitive to salicylic acid or suffering from poor blood circulation.
- Pregnant women and children: only use the device with healthcare supervision.
- For genital warts or in the genital area.
- Should not be used in the vicinity of the eyes nor in deep wounds with narrow openings.
- Skin damage will occur if the healthy skin come in contact with salicylic acid.

⚠ WARNINGS

- Only intended for external use
- Contact your doctor if your complaints do not subside
- Keep out of reach of young children
- Use by children, the elderly and less abled must be supervised by an adult.
- Avoid contact with clothes, textile, table-cloths or skin
- Do not re-use
- Dispose carefully in a manner preventing cross-contamination.

HOW TO USE

SIDE EFFECTS

Acetocaustin Wart Remover Plaster may cause some side effects, although not everyone will experience these side effects. The side effects that could occur when using Acetocaustin Wart Remover Plaster are irritated skin, burning sensation, skin redness or oversensitivity reactions. If any of these side effects becomes serious, or if you notice any side effects not listed in these instructions for use, please consult your doctor or pharmacist.

INTERFERENCE WITH MEDICATIONS

As far as we know, there are no interactions with any other medications.

CONTENTS OF ACETOCAUSTIN WART REMOVAL PLASTER

Acetocaustin Wart Removal Plasters are made of Moleksin Fabric full coated with adhesive substrate containing 40% Salicylic acid. One packaging contains 20 Acetocaustin Wart Removal Plasters.

STORAGE CONDITIONS

Store Acetocaustin Wart Removal Plasters at a temperature of 0-40°C in a dry and dark place in the original packaging and out of reach of young children.



The expiry date is stated on the bottom of the packaging.

For more information about Acetocaustin Wart Removal Plaster visit www.acetocaustin.com. For all product enquiries, please contact the distributor.

MEDICAL DEVICE:

Acetocaustin Wart Removal Plaster is a class IIa Medical Device according to the Directive 93/42/EEC.



DISTRIBUTOR
THEOTCLAB
Fred. Roeskestraat 115
1076 EE Amsterdam
The Netherlands

EC REP E C Rep Ltd,
5 Fitzwilliam Square East
Dublin 2, D02 R744, Ireland

Acetocaustin Wart Removal Plaster is exclusively produced by Pharmaplast s.a.e. for TheOTCLab.

Pharmaplast S.A.E
Amria, Free zone 23512
Alexandria, Egypt

EXPLANATION OF SYMBOLS

2. EXPLANATION OF SYMBOLS

The warning signs and symbols on the packaging and instructions for use are essential to ensure that you use this product safely and correctly and to protect you and others from injury. The symbols are explained in the table below:

| Symbol | Explanation |
|--------|---|
| | Symbol for the manufacturer |
| | Symbol that indicates to consult the instructions for use |
| | Symbol that indicates that caution is necessary when operating the device |
| | Symbol for the expiration date |
| | Symbol that indicates that the item is a medical device |
| | Symbol indicates to keep the product away from sunlight |
| | Indicates a medical device that is intended for one use. |
| | Keep out of reach of children |
| | Not for pregnant women or while breastfeeding |
| | Symbol for the Lot number |
| | The product shall be stored between 0°C and 40°C |
| | Indicates a medical device that should not be used if the package has been damaged or opened. |
| | Indicates the presence of natural rubber or dry natural rubber latex as a material of construction within the medical device or the packaging of a medical device which may cause allergic reactions. |
| | Authorized representative in the European Community / European Union |

